

FORA[®] Safety Lancets

ACS023

Safe capillary blood sampling with automatic retraction

FORA[®] Safety Lancets are designed for safe and easy capillary blood collection. The single-use lancets retract automatically and permanently after the puncture, allowing the controlled depth of the incision and protecting users from cross-infection.

Comfortable Features

- Tri bevel sharpened point
- Ready-to-use. No pre-loading or pre-assembling is needed.
- Patented internal dual-spring mechanism
- Unique ergonomic design



30G 1.6 mm
FINE



32G 1.6 mm
MICRO FINE



33G 1.6 mm
MICRO FINE

Safety Features

- **Single-Use**—Reduces risks of cross-infection
- **Safe**—Automatic retraction after the puncture
- **Easy to Use**—Twist and press
- **Accurate**—Controlled puncture depth
- **Stable Quality**—Sterilized, 5-year shelf life




26G 1.8 mm



28G 1.8 mm


Blood Sampling in Four Easy Steps

1. TWIST

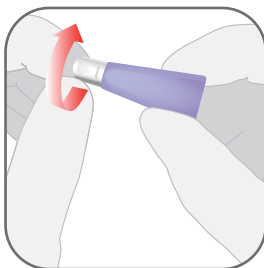


Twist off and discard the lancet cap.

2. PLACE

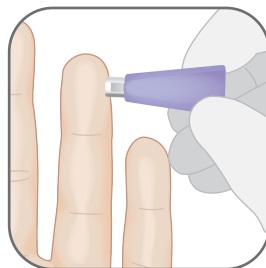

Place the FORA[®] Safety Lancet on the finger.

3. PRESS

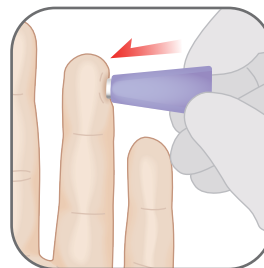

Press the platform firmly against the chosen site. The lancet will automatically retract into the lancet body.

4. DISPOSE



Dispose of the used lancet in a suitable sharps container.



FORA® Safety Lancets

Available in 50 and 100 pieces

FORA® Safety Lancets 30G
50 pcs ▶



FORA® Safety Lancets 30G
100 pcs ▶



Items	Color	Q'ty / box	Depth	Amount of bleeding
FORA® Safety Lancets 33G MICRO FINE		50 / 100 pcs	1.6 mm	
FORA® Safety Lancets 32G MICRO FINE		50 / 100 pcs	1.6 mm	
FORA® Safety Lancets 30G FINE		50 / 100 pcs	1.6 mm	
FORA® Safety Lancets 28G		50 / 100 pcs	1.8 mm	
FORA® Safety Lancets 26G		50 / 100 pcs	1.8 mm	

Used for blood glucose monitoring and other diagnostic tests in

- Diabetes clinics
- Hospitals
- Nursing homes
- Community health centers
- Blood donation centers
- Home use

ForaCare Suisse AG

Neugasse 55, 9000 St. Gallen, Switzerland

TEL : +41-71-220-1001

FAX : +41-71-220-1075

URL : www.foracare.ch

E-mail : chservice@foracare.com



www.foracare.ch