
FORA[®]

DUO ultima D40

Blood Glucose Plus Blood Pressure Monitoring System/
System zur Überwachung von Blutzucker und Blutdruck/
Système de Contrôle de la Glycémie et de la Pression Artérielle/
Sistema di Controllo della Glicemia e della Pressione Sanguigna/
Σύστημα παρακολούθησης σακχάρου αίματος και αρτηριακής πίεσης/
Sistema de supervisión de glucosa en sangre y presión arterial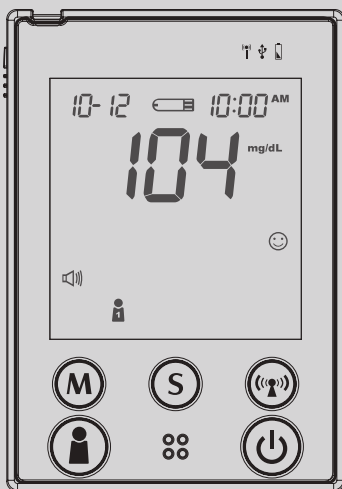

Owner's Manual / Bedienungsanleitung /
Manuel de l'utilisateur / Manuale del Proprietario /
Εγχειρίδιο χρήσης / Manual del propietario

Safety Information

Read the following *Safety Information* thoroughly before using the device.

- Use this device **ONLY** for the intended use described in this manual.
- Do **NOT** use accessories which are not specified by the manufacturer.
- Do **NOT** use the device if it is not working properly or damaged.
- Do **NOT** use the equipment where aerosol sprays are being used or where oxygen is being administered.
- Do **NOT** use under any circumstances on newborns or infants.
- This device does **NOT** serve as a cure for any symptoms or diseases. The data measured is for reference only. Always consult your physician to have the results interpreted.
- Before using this device to test blood glucose, read all instructions thoroughly and practice the test. Do all quality control checks as directed.
- Keep the device and testing supplies away from young children. Small items such as the battery cover, batteries, test strips, lancets and vial caps are choking hazards.
- Keep the equipment and its flexible cord away from hot surfaces.
- Do **NOT** apply the cuff to areas other than the place directed.
- Use of this instrument in a dry environment, especially if synthetic materials are present (synthetic clothing, carpets etc.) may cause damaging static discharges that may cause erroneous results.
- Do **NOT** use this instrument in close proximity to sources of strong electromagnetic radiation, as these may interfere with the correct operation.
- Proper maintenance and periodically calibration are essential to the longevity of your device. If you are concerned about your accuracy of measurement, please contact local customer service for help.

Important Information

- Severe dehydration and excessive water loss may cause readings which are lower than actual values. If you believe you are suffering from severe dehydration, consult a healthcare professional immediately.
- If your blood glucose results are lower or higher than usual, and you do not have symptoms of illness, first repeat the test. If you have symptoms or continue to get results higher or lower than usual, follow the treatment advice of your healthcare professional.
- Use only fresh whole blood sample to test your blood glucose.
- Using other substances will lead to incorrect results.
- If you are experiencing symptoms that are inconsistent with your blood glucose test results and you have followed all instructions described in this owner's manual, call your healthcare professional.
- We do not recommend using this product on severely hypotensive individuals or patients in shock. Please consult the healthcare professional before use.
- The measurement unit used for indicating the concentration of blood or plasma glucose can have mg/dL or mmol/L. The approximate calculation rule for conversion of mg/dL in mmol/L is:

mg/dL	Divided by 18	= mmol/L
mmol/L	Times 18	= mg/dL


Introduction

Intended Use

This system is a 2 in 1 system designed to measure blood glucose outside the human body (in vitro diagnostic use) and to measure blood pressure non-invasively. It is intended for use in the home and in clinical settings as an aid to monitoring the effectiveness of diabetes control.


It is intended to be used for the quantitative measurement of glucose (sugar) in fresh whole blood samples drawn from the finger, palm, forearm, upper arm, calf and thigh. It should not be used for the diagnosis of diabetes, or testing on newborns. This system is also intended to be used to measure non-invasively the systolic and diastolic blood pressure and pulse rate of an adult individual at home. The blood pressure is measured by using an inflatable cuff wrapped around the arm.

Product Overview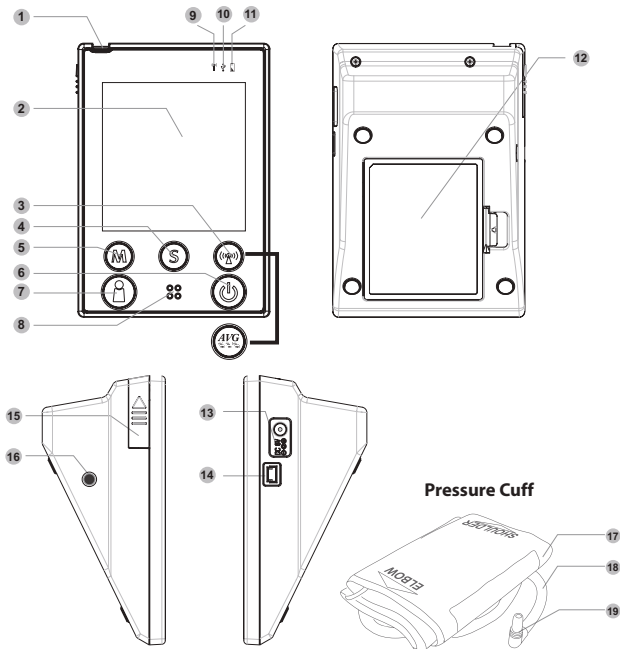

No.	Item	Description
1	Test Slot	Insert test strip here to turn the device on for testing.
2	Display Screen	<ul style="list-style-type: none"> • During power off: Display the date and time. • During power on: Display the selected setting or test result.
3	Average Button (For D40a model)	Press to measure your blood pressure in average mode.
	RF Button (For D40b/D40g model)	Press to activate the wireless data transmission.
4	S Button	Press to enter and confirm the settings.
5	M Button	Press to enter the device memory.
6	Power Button	Press to turn on the device and start a single blood pressure measurement.
7	User Selection Button	Press to select the desired user number.
8	Speaker	Emit sound or voice broadcast.
9	RF Indicator	Indicate a wireless data transmission status.
10	USB Indicator	Indicate a USB connection status.
11	Power Indicator	Indicate the power status of the device.
12	Battery Compartment	Open the cover to install the batteries.
13	AC/DC Adapter Port	Connect to a power supply.
14	USB Port	Connect to a micro USB cable.
15	Strip Ejector	Push upward to eject the used strip.
16	Air Jack	Connect to an air plug.
17	Pressure Cuff	Measure blood pressure.
18	Air Tube	Inflate the pressure cuff.
19	Air Plug	Connect to air jack.

Screen Display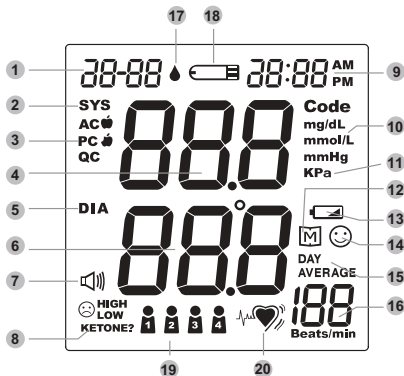

No.	Item
1	Date
2	Systolic Pressure Symbol
3	Measurement Mode
4	Systolic Pressure Value
5	Diastolic Pressure Symbol
6	Diastolic Pressure Value
7	Volume Symbol
8	Special Message
9	Time
10	Unit for Blood Glucose

No.	Item
11	Unit for Blood Pressure
12	Memory Mode Symbol
13	Battery Symbol
14	Result Indicator
15	Day Average Result
16	Pulse Rate
17	Blood Drop Symbol
18	Test Strip Symbol
19	User Number
20	Heart Symbol - for IHB Detection (Irregular Heart Beat)

Voice Function

This device is equipped with voice function to guide you through the process of blood glucose and blood pressure testing.

The following table tells you when and what the device “speaks”.

Situation	Spoken sentence(s)
When the device is turned on to begin a blood pressure measurement.	Thank you for using this product. The user's number is one, please select the user's number. The user's number is (number), please relax during measurement.
When room temperature is outside operating range of 10°C to 40°C (50°F to 104°F).	Room temperature out of range, unable to measure.
When the glucose test is completed (the result appears on display).	The blood glucose is (number) milligram per deciliter.
When the device is in the setting mode.	You are now in setting mode. The year is (number).
	The month is (month).
	The day is (number).
	The hour is (number).
	The minute is (number).
	The time is (12/24) hour mode. Voice volume is (number).
When the device is in the setting mode.	Delete memory record function, press M key to clear all the records.
	The year is (number). The month is (month). The day is (number). The hour is (number). The minute is (number). The time is (12/24) hour mode. Voice volume is (number).
	Have a nice day.
When the test strip is inserted into the device.	Thank you for using this product. Test strip is inserted. Today's day is (number), the time is (number), the user's number is one, please apply blood onto the test strip.
	Now testing.
	The user's number is one. Your blood glucose is (number) milligram per deciliter.

Situation	Spoken sentence(s)
When the glucose test result is outside measurement range of 20 to 600 mg/dL (1.1 to 33.3 mmol/L).	The blood glucose is out of range, unable to measure.
When power is insufficient.	Battery is dead, please replace.
When the device is in blood glucose measurement mode.	You are now in testing mode.
	This is before meal measurement.
	This is after meal measurement.
	You are now in control solution mode.
When the blood pressure test is completed (the result appears on display).	Your systolic blood pressure is (number) millimeter of mercury, the diastolic blood pressure is (number) millimeter of mercury and the heart rate is (number) beats per minute.
When the device is in memory mode.	Thank you for using this product. Today's date is (number), the time is (number), you are now in memory mode, the user's number is one, please select the user's number.
When the device is in memory mode.	Memory record for (time), your blood glucose is (number) milligram per deciliter. Your memory record for (time). Your systolic blood pressure is (number) millimeter of mercury, the diastolic blood pressure is (number) millimeter of mercury and the heart rate is (number) beats per minute.
When the device is in day average mode.	Thank you for using this product. You are now in day average mode, the user's number is one, please select the user's number.
	The user's number is (number), the 7-,14-,21-,28-,60-,90- day blood glucose average is (number)

Getting Started

Initial Setup

Before using the device for the first time or after you changed the batteries, do the following initial setup:

Step 1: Enter the Setting Mode


Press and hold **S** until the display turns on.

Step 2: Configuring the Settings

(Date, Time, Time Format, Volume Level, Auto-send and Memory Deletion)


Press **M** repeatedly to adjust the value or enable/disable the setting. Then press **S** to confirm the setting and switch to another field.

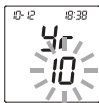
Set [Year]


Set [Month]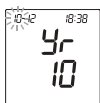

Set [Day]

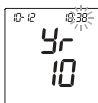Set [Time
Format]

Set [Minute]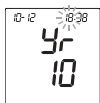

Set [Hour]

Set [Volume
Level]

Set [Auto-Send]

Memory
Deletion

After all settings are complete, the device will automatically repeat all settings before shutdown.

Note:

- During memory deletion, do the following:
 - ✓ To keep all saved results, press **S** to keep the saved result.
 - ✓ To delete all results, press **M** twice to delete all memory.
 - ✓ To delete memory individually, press **I** to select specific user. Press **M** twice to delete all results for the specified user.
- If the device is idle for 3 minutes during the setting mode, it will turn off automatically.

User Number Selection

This system provides up to four (4) users for both measuring functions. Each user's test results are stored separately under each user number.


1. Press **I** repeatedly to select the desired user number.

The flashing icon indicates the currently selected user.


2. Press **S** to confirm the selection.

The device will repeat the selected user number and then turn off automatically.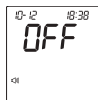


Testing Your Blood Pressure

Fitting the Cuff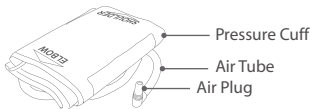

1. Connect the air plug of the tubing to the air jack of the device.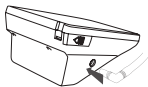


2. Assemble the cuff as shown in the illustration.

The smooth surface should be inside the cuff loop and the metal D-ring should not touch your skin.


Note:

- Before using the cuff, be sure the cuff size fits with your upper arm circumference.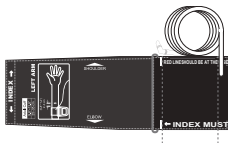

3. Stretch your left (right) arm in front of you with your palm facing up.

Slide and place the cuff onto your arm with the air tube and artery mark region (in red) toward the lower arm.

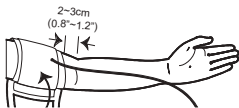Wrap and tighten the cuff above your elbow. The red line on the edge of the cuff should be approximately 0.8 to 1.2 inches (2 to 3 cm) above your elbow. Align the tube over the main arteries on the inside.

4. Leave a little free space between the arm and the cuff. You should be able to fit 2 fingers between them.
5. Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.

Proper Measurement Position

1. Sit down for at least 10 minutes before measuring.
2. Place your elbow on a flat surface. Relax your hand with the palm facing up.
3. Make sure the cuff is about the same height as the location of your heart.
4. After the device is turned on, the cuff will begin to inflate automatically. Remain still and do not talk or move during the measurement.

Measuring Blood Pressure

This device provides you two different ways to measure your blood pressure. When the device detects an irregular heart beat, the pulse rate symbol is shown as "♥" instead of "♥". If the problem still persists, please consult your doctor.


Important!

Always apply the pressure cuff before turning on the device.

A. Single Measurement

Perform an individual blood pressure measurement.

1. Press **⏻**. All the LCD symbols will appear.
2. Press **1** to select the desired user number and then press **S**. Then the cuff will begin to inflate automatically.

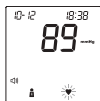A pulse is detected.

Displays the systolic pressure, diastolic pressure, and pulse rate.

3. Press **⏻** to turn the device off.

Note:

- If the device is idle for 3 minutes, it will turn off automatically.

B. Average measurement


Automatically perform three (3) consecutive blood pressure measurements and display the final average result.

1. Enter the Average mode.

Press and hold **⏻** for 3 seconds (for D40b/D40g model), or just press **AVG** (for D40a model).

2. Press **1** to select the desired user number and then press **S**. Then the cuff will begin to inflate automatically.

After the first measurement is complete, the device will start counting down before the second measurement begins. The device will take three (3) measurements consecutively with an interval of 20 seconds.

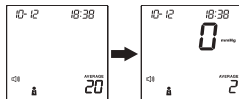After taken three measurements, the results are averaged to produce the blood pressure measurement.

3. Press **⏻** to turn the device off.

Control Solution Testing

FORA Control Solution contains a known amount of glucose that reacts with test strips and is used to ensure your device and test strips are working together correctly.

Do a control solution test when:

- ✓ you first receive the device.
- ✓ at least once a week to routinely check the device and test strips.
- ✓ you begin using a new vial of test strips.
- ✓ you suspect the device or test strips are not working properly.
- ✓ your blood glucose test results are not consistent with how you feel, or if you think the results are not accurate.
- ✓ you have dropped or think you may have damaged the device.

To perform the control solution test, do the following:

1. Insert the test strip into the test slot of the device.
Wait for the device to display the test strip and blood drop symbol.
2. Press **F** to select the desired user number.
3. Press **M** to mark this test as a control solution test.

With "QC" displayed, the device will store your test result in memory. If you press again, the "QC" will disappear and this test is no longer a control solution test.

Important!



- When doing the control solution test, you have to mark it so that the test result will not be stored in the memory. Otherwise, it will mix up the blood glucose test results with the control solution test results in memory.

4. Apply the control solution.

Shake the control solution vial thoroughly before use. Squeeze out a drop and wipe it off, then squeeze another drop and place it on the tip of the vial cap.

Hold the device to move the absorbent

hole of test strip to touch the drop. Once the confirmation window fills completely, the device will begin counting down.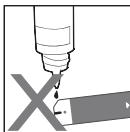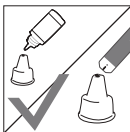


Note:

- To avoid contaminating the control solution, do not directly apply control solution onto a strip.

5. Read and compare the result.


After counting down to 0, the test result of control solution will appear on the display. Compare this result with the range printed on the test strip vial and it should fall within this range. If the test result is out of range, read the instructions again and repeat the control solution test.

(110 mg/dL = 6.1 mmol/L, 109~165 mg/dL = 6.0~9.1 mmol/L)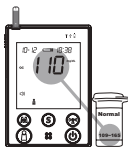

Testing Your Blood Glucose

Test Strip Appearance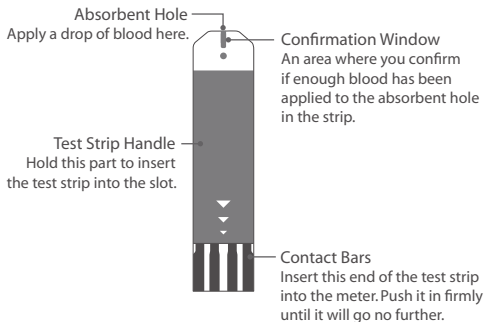

Inserting a Test Strip

Insert the test strip as shown in the illustration.


Important!

The front side of test strip with FORA Logo should face up when inserting test strip. Test results might be wrong if the contact bar is not fully inserted into the test slot.

Preparing the Lancing Device

1. Remove the cap.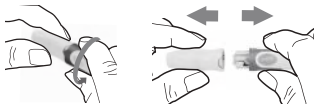

2. Insert a new lancet firmly into the white lancet holder cup.

3. Remove the protective disk on the lancet.

Hold the lancet firmly in place and twist off the protective disk.

4. Replace the cap until it snaps or clicks into place.
5. Rotate the dial to set the desired lancing depth.
6. Pull the cocking control out until the orange bar appears on the release button window.


Obtaining the Blood Sample

Please follow the suggestions below before obtaining a drop of blood:

- Wash and dry your hands before starting.
- Select the puncture site either at fingertips or another body parts.
- Clean the puncture site using cotton moistened with 70% alcohol and let it air dry.
- Rub the puncture site for about 20 seconds before penetration.
- Use a clear cap (included in the kit) while setting up the lancing device.
Wait for the device to display the test strip and blood drop symbol.

❖ Blood from the fingertip

1. Press the lancing device's tip firmly against the lower side of your fingertip.
2. Press the release button to prick your finger, then a click indicates that the puncture is complete.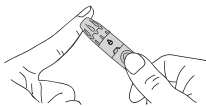


❖ Blood from sites other than the fingertip

Important!

- Avoid lancing areas with obvious veins to avoid excess bleeding.

1. Replace the lancing device cap with the clear cap.
2. Pull the cocking control out until the orange bar appears on the release button window.

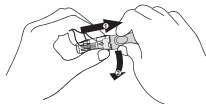Note:

- Choose a different spot each time you test. Repeated punctures at the same spot may cause soreness and calluses.
- Please consult your health care professional before you begin AST (Alternative Site Testing).
- It is recommended to discard the first drop of blood as it might contain tissue fluid, which may affect the test result.

Measuring Blood Glucose Test

1. Insert the test strip into its slot.

Wait for the device to display the test strip and blood drop symbol.


2. Select the appropriate measuring mode by pressing **M**.

- **General Tests** (not display) - any time of day without regard to time since last meal.
- **AC**  - no food intake for at least 8 hours.
- **PC**  - 2 hours after a meal.
- **QC** - testing with the control solution.

3. Obtain a blood sample.

Use the pre-set lancing device to puncture your desired site. After penetration, discard the first drop of blood with a clean tissue or cotton. Gently squeeze the punctured area to obtain another drop of blood. Be careful NOT to smear the blood sample.


The volume of blood sample must be at least 0.5 microliter (μL) of volume.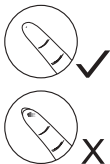

4. Press **F** to select the desired user number.

5. Apply the blood sample.

Hold the blood drop to touch the absorbent hole of the test strip. Blood will be drawn in and after the confirmation window is completely filled, the device begins counting down.

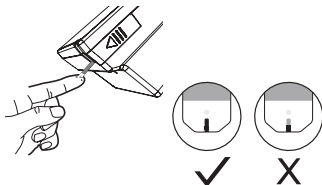6. Read your result.

The result of your blood glucose test will appear after the device counts down to 0. This result will automatically be stored in the memory.


(104 mg/dL = 5.7 mmol/L)

Disposing Used Test Strip and Lancet

To remove the used test strip, simply push the **Strip Ejector** button upward to eject the used test strip. The device will automatically turn off after the test strip is ejected.

To remove the used lancet, remove the lancet from the lancing device after you have finished testing.

Discard your used strip and lancet properly in a puncture resistant container.

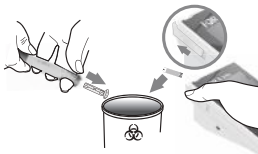Reviewing Test Results

Your device stores the 864 most recent test results along with respective dates and times in its memory.

To review all test results, do the following:

1. Press **M**. The "M" icon appears on the screen.
2. Press **I** to select the desired user number.
3. Press **M** to review the test results stored in the device. Press **M** to search the result forwards, and press **S** to search the result backwards.

(109 mg/dL = 6.0 mmol/L)

To review the day-average test results, do the following:

1. Press and hold **M** for 3 seconds until the "DAY AVERAGE" icon appears on the screen. Your blood glucose 7-day average result will appear on the display.
2. Press **I** to select the desired user number.

(115 mg/dL = 6.3 mmol/L)

3. Press **M** to review your 14-, 21-, 28-, 60- and 90-day average.
4. Press **S** to review your blood pressure 7-, 14-, 21-, 28-, 60- and 90-day average.

Note:

- If you wish to review the results of different user numbers or the blood glucose and blood pressure for each user number, press **F** or **S**.
- Press **Q** to exit the memory mode or leave it without any action for 3 minutes. The device will turn off automatically.
- If using the device for the first time, the “---” icon appears when you recall the test results or review the average result. This indicates that there is no test result in the memory.
- Control solution results are **NOT** included in the day average.
- The blood glucose test results of **Lo** and **Hi** will **NOT** be calculated in the day average results.

Appendix

Transferring Data

There are three types of connection to transfer results, please check your meter box for the transmission method of your meter. For USB Continua, Bluetooth or GPRS methods, please contact service providers for more information.

1. Data Transmission Via Cable (For D40a model only)

a. USB

i. Install the software on your computer


Download Health Care System Software and instruction manual provided on ForaCare Suisse AG, website: <http://www.foracare.ch> to your computer. Follow the instructions to install the software in your computer.

ii. Connect the device with your computer using a USB cable

“USB” will appear on the display and the USB indicator will light up in green, indicating that the meter is ready to transmit data.

iii. Transfer data to your computer

Follow the software on-screen instructions to transmit data. Data transmitted will include results with date and time. Remove the cable and the device will automatically turn off.

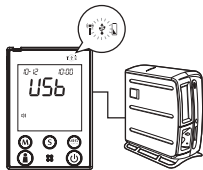b. Continua

Please identify the device (agent) connects to manager side (host, hub or gateway) has followed the Continua certification, and please contact your service provider for account registration and procedure.

2. Data Transmission Via Bluetooth (For D40b model only)

a. Type 1

i. Activate Bluetooth


To enable the Bluetooth function, do one of the following:

- with the device turned off, press  to turn on the Bluetooth function.
- if Auto-send is set to **On**, the device will turn on the Bluetooth function after the system completes a measurement.

ii. Pair with your computer

Your computer will prompt you to enter the the Bluetooth pin code. Enter the correct pin code (111111). "PCL" will appear on the display and the RF indicator will light up in blue.

When the RF indicator is blinking, this indicates that the device is pairing mode. Once the connection is established, the RF indicator shows a steady blue light, and it is ready for data transmission.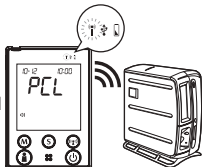

BT Indicator	Status	BT Indicator	Status
Flash	Connectable for pairing.	Lit Solid	The connection is established.

iii. Transfer data to your computer

The transmission procedure is controlled by software, the device will turn off by software after completion of data transmission. If the device is still on, press  and the device will turn off.


b. Type 2

i. Activate Bluetooth

To enable the Bluetooth function, do one of the following:

(a) Pairing mode:

With the device turned off, press and hold  for 5~10 seconds until the screen shows "PCL" and then "dEL" will flash twice, the RF indicator will light up in blue and blink fast. For the first time connection, your computer will prompt you to enter the pin code (111111) and the pairing starts.

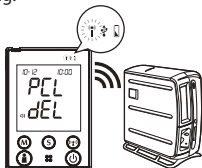(b) Connection mode:

- with the device turned off, press  to turn on the Bluetooth function.
- if Auto-send is set to **On**, the device will turn on the Bluetooth function after the system completes a measurement.

After measurement or short press , the RF indicator will blink slowly and the device is in connection mode, only paired host can connect to the device. When the RF indicator shows a steady blue light, the device is online and ready for data transmission.

BT Indicator	Status	BT Indicator	Status
Flash Fast	Pairing mode.	Lit Solid	The connection is established.
Flash Slowly	Connection mode.		

ii. Transfer data to your computer

The transmission procedure is controlled by software, the device will turn off by software after completion of data transmission. If the device is still on, press  and the device will turn off.

c. Continua

With regard to how to process the pairing, please refer to Bluetooth Type 2.

Note:

- Make sure your PC has Bluetooth before transmitting the data and the device is within the receiving range.
- When the device is connected to the PC, it is unable to perform any test.

3. Data transmission via GPRS (For Fora D40g only)

Before initiating GPRS transmission, insert the SIM card at the bottom of the device battery compartment to transmit data. Please contact service providers for more information on SIM card, server information and data transmission.


i. Initiate GPRS transmission

To initiate GPRS transmission, do one of the following:

- with the device turned off, press  to initiate GPRS transmission of batch upload for current user with test results which have not been uploaded,
- if Auto-send is set to **On**, the device will initiate GPRS after measurement to send the current new data,
- in memory mode, press  to initiate GPRS to upload data shows on the screen.

ii. During transmission

With the device turned off, press  to initiate the data transmission from the device. "PCL" will appear on the display first and the RF indicator will light up in blue. Then the test result under transmission will show on the screen.

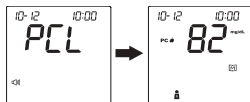When the RF indicator is blinking in blue, the data transmission has started. When the RF indicator shows an orange light, the GPRS service is online.

Press  and the device will turn off.

Note:

- If the SIM card is not inserted, the "PCL" will start blinking, and the device will turn off automatically after blinking for 30 seconds. If the problem still persists, contact the place of purchase for help.
- When the data transmission is in progress, the device is unable to perform any test.
- **Signal Quality Check**

To begin the quality check, press and hold  for 10-15 seconds when the device is off. Wait until PCL appears on the screen. The number displayed in the middle of the screen indicates the signal quality of GPRS at where you are. For example, the signal quality in illustration below displays 16.

The greater the number, the better signal you have. We recommend that the signal quality should be above ten (10) when you perform the transmission.

If you receive poor signal quality, take the device to a spot with a better reception and try again. Keep away from microwave or other devices that may interrupt the signal.


Maintenance

Changing Battery

When the battery is low, the "

To change the batteries, do the following:

1. Press the edge of the battery cover and lift it up to remove the cover.
2. Remove the old batteries and replace with four 1.5V AA size alkaline batteries.
3. Close the battery cover.

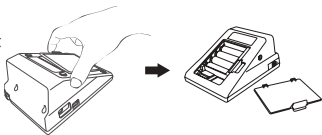When you will hear a "beep" sound, this indicates the batteries are inserted correctly.

CAUTION
RISK OF EXPLOSION IF BATTERY IS REPLACED
BY AN INCORRECT TYPE.
DISPOSE OF USED BATTERIES ACCORDING TO
THE INSTRUCTIONS.

Note:

- Replacing the batteries does not affect the test results stored in memory.
- Keep away these batteries from small children. If swallowed, promptly seek medical assistance.
- Batteries may leak chemicals if unused for a long time. Remove the batteries if you are not going to use the device for an extended period.
- Properly dispose of the used batteries according to your local environmental regulations.

Recharging the Battery (For FORA D40g Only)

The power adapter can be used as the battery charger when you need to recharge Li-ion battery. To recharge the 3.7V Li-ion battery, connect AC adapter to the monitor and a power source.

The power indicator will light up in green, which indicates the battery is undergoing a recharge. It should take approximately 5 hours to be fully charged.

If the power indicator starts blinking in green, this indicates the recharge is nearly finished. The power indicator will diminish when the recharge completes.

Using an AC Adapter

As an alternative, you can use an AC adapter to provide the power supply. However, when alkaline batteries are used for FORA D40a and FORA D40b, these batteries cannot be recharged with the device by connecting AC adapter.

To use an AC adapter, do the following:

1. Connect AC adapter plug to AC adapter jack of the device.
2. Plug AC adapter power plug into an electrical outlet.

CAUTION!

- For D40g, the Li-ion battery must be recharged with the FORA D40g monitor by connecting AC adapter to an electrical outlet. Only this 3.7V rechargeable Li-ion battery (battery pack no. S14500 1S4P) can be recharged with FORA D40 by the connection of AC adapter. Any other kind of rechargeable battery is not allowed or it may damage the monitor.

Caring for Your Device

To clean the device exterior, wipe it with a cloth moistened with tap water or a mild cleaning agent, then dry the device with a soft dry cloth. Do NOT flush with water.

- Do NOT use organic solvents to clean the device.
- Do NOT wash or iron the pressure cuff.
- Always store or transport the device in its original storage case.
- Avoid dropping and heavy impact.
- Avoid direct sunlight and high humidity.

Device Storage

- Storage condition: -20°C to 60°C (-4°F to 140°F), below 95% relative humidity.
- Always store or transport the monitor in its original storage case.
- Avoid dropping and heavy impact.
- Avoid direct sunlight and high humidity.

Caring for Your Test Strips

- Storage condition: 4°C to 40°C (39.2°F to 104°F), below 85% relative humidity. Do not freeze.
- Store your test strips in their original vial only. Do not transfer to other container.
- Store test strip packages in a cool and dry place. Keep away from direct sunlight and heat.
- After removing a test strip from the vial, immediately close the vial cap tightly.
- Touch the test strip with clean and dry hands.
- Use each test strip immediately after removing it from the vial.

- Write the opening date on the strip vial label when you first opened it. Discard remaining test strips after 3 months.
- Do not use test strips beyond the expiry date. This may cause inaccurate results.
- Do not bend, cut, or alter a test strip in any way.
- Keep the strip vial away from children since the cap and the test strip may be a choking hazard. If swallowed, promptly see a doctor for help.

For further information, please refer to the test strip package insert.

Reference Values

Blood Glucose

The meter provides you with plasma equivalent results.

Time of day	Normal plasma glucose range for people without diabetes (mg/dL)
Fasting and before meal	Less than 100 mg/dL (5.6 mmol/L)
2 hours after meals	Less than 140 mg/dL (7.8 mmol/L)

Source: American Diabetes Association (2012). *Clinical Practice Recommendations. Diabetes Care, 35 (Supplement 1): S1-100.*

Please consult your doctor to determine a target range that works best for you.

Blood Pressure

Clinical studies show that adult diabetes is often accompanied by elevated blood pressure. People with diabetes can reduce their heart risk by managing their blood pressure along with diabetes treatment^{*1}.

Definitions and Classification of blood pressure levels according to 2007 ESH-ESC Practice Guidelines for the Management of Arterial Hypertension:

Category	Systolic (mmHg)		Diastolic (mmHg)
Optimal	< 120	and	< 80
Normal	120 –129	and/or	80–84
High normal	130 –139	and/or	85–89
Grade 1 hypertension	140 –159	and/or	90– 99
Grade 2 hypertension	160 –179	and/or	100–109
Grade 3 hypertension	≥ 180	and/or	≥ 110
Isolated systolic hypertension	≥ 140	and	< 90

Isolated systolic hypertension should be graded (1, 2, 3) according to systolic blood pressure values in the ranges indicated, provided that diastolic values are < 90mmHg.

Source: The European Society of Hypertension and European Society of Cardiology Task Force Members. 2007 ESH-ESC Practice Guidelines for the Management of Arterial Hypertension. *J Hypertens 2007; 25: 1751-1762.*

^{*1}: American Diabetes Association: The Diabetes-Heart Disease Link Surveying Attitudes, Knowledge and Risk (2002)

Symbol Information


Symbol	Referent
	For in vitro diagnostic use
	Read instructions before use
	Use by/ Expiry date
	Batch code
SN	Serial number
	Green Dot: German dual waste disposal system
	CE Mark

Symbol	Referent
	Do not reuse
	Type BF equipment
	Temperature limitation
	Biological Risks
	Manufacturer
	Caution, consult accompanying documents

Troubleshooting

If you follow the recommended action but the problem persists, or error messages other than the ones below appear, please call your local customer service.


Error Message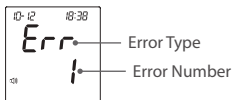

Error Message	Cause	What To Do
Err 1	Inflation or pressure error.	Please contact local customer service for help.
Err 2		Refit cuff tightly and correctly. Relax and repeat the measurement. If the error still remains, please contact local customer service for help.
Err 3	Cuff pressure is too high.	
Err 4 or Err 5	No pulse rate detected or the pulse rate is too low.	Turn off the device and start over. Follow the instructions and try again.
Err 8	Forced to stop the blood pressure measurement after repump for 2 times.	
Err 15	Data corrupted.	Follow the instructions and try again. If problem persists, contact the local customer service for assistance.

Error Message	Cause	What To Do
E-U	Strip has been used.	Repeat the test with a new strip.
E-d	Problem in operation.	Review the instructions and repeat the test with a new strip.
E-t	Environmental temperature is outside the system's operational range.	System operational range is 10°C to 40°C (50°F to 104°F). Repeat the test after the device and test strip have reached the above temperature.

Error Message of D40g GPRS Transmission

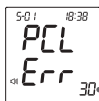30 — Error Number

Error No.	Cause	What To Do
30	Unable to connect to the service	Please start over and try again.
31	Incorrect PIN number of SIM card	Check if the entered PIN number of your SIM card is correct and try again. Reset the PIN number if necessary. If problem persists, please contact the customer service for assistance.
32	Incorrect PIN 2 number of SIM card	
33 / 34	Locked SIM card	
38	Unfinished registration	Complete the registration. Try again after you have successfully registered.
39	Problem of reading SIM card	Check if the SIM card is correctly placed and try again. If problem persists, please contact the customer service for assistance.
40	No response from the server	Turn off the device and reconnect to the service. Change to a spot with a better quality of connection and try again.
41	Service error	
42	Failure connecting to the service	If problem still persists, please contact local customer service for assistance.
43	Connection overtime	Turn off the device and try again to reconnect to the service.
44	Command failure	Turn off the device and try again. If problem still persists, please contact local customer service for assistance.
45	Data upload error	
48 / 49	Problem of connecting to the service	

Blood Glucose Measurement

Symptom	Cause	What To Do
The device does not display a message after inserting a test strip.	Test strip inserted upside down or incompletely.	Insert the test strip with contact bars end first and facing up.
	Defective device or test strips.	Please contact customer service.
The test does not start after applying the sample.	Insufficient blood sample.	Repeat the test using a new test strip with larger volume of blood sample.
	Defective test strip.	Repeat the test with a new test strip.
	Sample applied after the device is automatically turned off.	Repeat the test with a new test strip. Apply sample only when flashing "●" appears on the display.
	Defective device.	Please contact customer service.
The control solution testing result is out of range.	Error in performing the test.	Read instructions thoroughly and repeat the test again.
	Control solution vial was poorly shaken.	Shake the control solution vigorously and repeat the test again.
	Expired or contaminated control solution.	Check the expiration date of the control solution.
	Control solution that is too warm or too cold.	Control solution, device, and test strips should be at room temperature (20°C to 25°C / 68°F to 77°F) before testing.
	Defective test strip.	Repeat the test with a new test strip.
	Device malfunction.	Please contact customer service.

Blood Pressure Measurement

Symptom	Cause	What To Do
Nothing is displayed after pressing the Power button.	Batteries incorrectly installed or no battery is installed.	Check that the batteries are correctly installed.
The heart rate is higher/lower than user's average.	Movement during measurement	Repeat measurement
	Measurement taken just after exercise.	Rest at least 30 minutes before repeating measurement
The result is higher/lower than user's average measurement.	May not be in correct position while measuring.	Adjust to the correct position to measure.
	Blood pressure naturally varies from time to time.	Keep in mind for next measurement.
	Defective test strip.	Repeat the test with a new test strip.
	Device malfunction.	Please contact customer service.

Specifications

System performance	
Power Source	Four 1.5V AA alkaline batteries or one 3.7V rechargeable Li-ion battery
Device Dimensions (w/o cuff)	147 (L) x 105 mm (W) x 80 mm (H), 500g with batteries
Memory	Maximum 864 memory records
Power Saving	Automatic power off if system idle for 3 minutes (normal mode) or 5 minutes (RF mode).
System Operating Condition	10°C to 40°C (50°F to 104°F), below 85% RH
Device Storage/Transport Condition	-20°C to 60°C (-4°F to 140°F), 5 - 95% RH
Power Supply Input	DC +6V / 1A (max) via Power Plug
Blood glucose measurement performance	
Measurement Unit	mg/dL or mmol/L
Linear Range	20 to 600 mg/dL (1.1 to 33.3mmol/L)
Precision	±5 % (CV)
Accuracy	±15 mg/dL (0.83 mmol/L) when glucose < 75mg/dL (4.2 mmol/L)
	±20% when glucose ≥ 75mg/dL (4.2 mmol/L)
Ketone Warning	glucose value is over 240 mg/dL (13.3 mmol/L)
Blood pressure measurement performance	
Pressure Range	0 - 300 mmHg
Heart Rate Range	40 -199 beat per minute
Measurement Unit	mmHg
Systolic Measurement Range	55 mmHg -255 mmHg
Diastolic Measurement Range	25 mmHg -195 mmHg
Maximum Inflation Pressure	280 mmHg
Accuracy of Pressure	±3 mmHg or ±2% of reading
Accuracy of Pulse Rate	±4% of reading

This device has been tested to meet the electrical and safety requirements of: IEC/EN 60601-1, IEC/EN 60601-1-2, IEC/EN 61010-1, IEC/EN 61010-2-101, IEC/EN 61326-2-6, EN 301 489-17, EN 300 328.

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Quelle: American Diabetes Association (2012). *Clinical Practice Recommendations. Diabetes Care*, 35 (Supplement 1): S1-100.

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Quelle: *The European Society of Hypertension and European Society of Cardiology Task Force Members. 2007 ESH-ESC Practice Guidelines for the Management of Arterial Hypertension. J Hypertens 2007; 25: 1751-1762.*

*1: American Diabetes Association: The Diabetes-Heart Disease Link Surveying Attitudes, Knowledge and Risk (2002)

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Source : *American Diabetes Association (2012). Clinical Practice Recommendations. Diabetes Care, 35 (Supplement 1): S1-100.*

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Source : *The European Society of Hypertension and European Society of Cardiology Task Force Members. 2007 ESH-ESC Practice Guidelines for the Management of Arterial Hypertension. J Hypertens 2007; 25: 1751-1762.*

*1 : American Diabetes Association: The Diabetes-Heart Disease Link Surveying Attitudes, Knowledge and Risk (2002)

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