# FORA MT86

## DIGITAL THERMOMETER INSTRUCTION MANUAL

Version 2.0, 10/2020  
311-0000000-052

EN

### Dear Customer,

Thank you for choosing the new FORA digital thermometer. As this manual provides important information regarding proper use of the product, please read the instructions carefully before use.

**Warning:** The product is not a toy. Never leave the thermometer unattended and keep away from children.

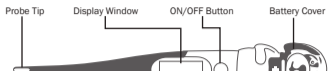
### Important Safety Information

- The instrument may only be used for measuring body temperature!
- Pay attention when using the thermometer on children and the elderly. The thermometer should only be used under the supervision of an adult.
- Protect the instrument from any impact!
- Do not store the unit where it will be exposed to direct sunlight, dust or humidity. Avoid extreme temperatures.
- Avoid ambient temperatures above 60°C. NEVER hold the instrument!
- Store the unit in its protective case when not in use.
- We recommend that this instrument be tested for accuracy every two years or after mechanical impact (e.g. when dropped). Please contact ForCare Suisse AG distributor to arrange the test.

### What is a "Normal" temperature?

Although the generally accepted "normal" temperature reading is 37.0°C / 98.6°F, temperature readings can vary from 36.1°C / 97.0°F to 37.2°C / 99.0°F and may still be considered "normal". Variations in temperature can be attributed to activities such as exercise, smoking, eating or drinking. Your temperature is lower in the morning than in the afternoon. Other variations may occur due to the location of the temperature reading. While the oral temperature follows the guidelines previously described, a rectal temperature is generally 0.5°C / 1.0°F higher. Conversely, an axillary (under the arm) temperature will be 0.5°C / 1.0°F lower.

### Digital Thermometer Overview

\* Please note that the animal design on the thermometer may be different from the drawing depicted above.

### Cleaning Instructions

Before and after each use, clean the thermometer with a soft cloth and isopropyl alcohol diluted in water, or wipe it with COLD soapy water. Please Do NOT soak it in the water for an extended period of time or sterilize it by boiling, gas or steam autoclaves.

### How to take your temperature?

- Press the ON/OFF button to activate it. The unit will beep and "188.8°C" will be displayed. This LCD display test will go on for about 2 seconds.
- "37.0°C" will appear on the display.
- As soon as "Lo" and a flashing "C" appear on the display, the thermometer is ready for measurement.

**Note:** If the room temperature is above 32.0°C, the room temperature will be displayed instead of "Lo" °C. If the room temperature is ≥ 43.0°C the display shows "Hi" °C. 4. Choose the preferred measuring method. Place the thermometer probe in the chosen location. When taking a measurement, the current temperature is continuously displayed and the "C" symbol flashes. 5. A beep will sound when the measurement is complete. 6. Press the ON/OFF button to turn the thermometer off. (The unit will automatically switch off when idle for approximately 10 minutes)

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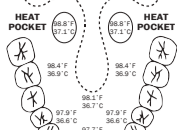
### Taking an oral temperature

Place the probe tip of the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The probe tip must be in good contact with the tissue. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/ exhaled air. Approximate temperature reading time: 10 - 30 seconds. It is recommended regardless of the number of beeps.

### Taking an axillary temperature (under the arm)

For more reliable results, we recommend measurements of oral or rectal temperatures. Wipe the underarm with a dry towel and place the probe tip under the arm so that the tip is touching the skin with the thermometer perpendicular to the body. Position the arm across the chest so the probe tip is well covered by the arm. Approximate temperature reading time: 3-5 minutes. It is recommended regardless of the number of beeps. **Note:** Axillary temperatures are generally 0.5°C / 1.0°F lower than oral readings.

### Taking a rectal temperature

This is the most reliable method of measuring body temperature and it is especially suitable for infants and small children. Lubricate the probe with a water-soluble lubricant. Gently insert the probe into the rectum about 1 cm and switch the unit on. Approximate measuring time: 10 seconds. **Note:** Rectal temperatures are generally 0.5°C higher than oral readings.

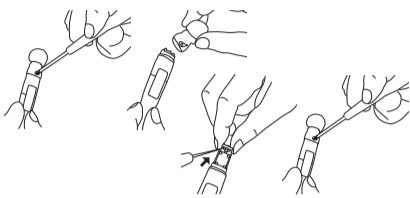
### Memory

The thermometer stores the last memory reading. If you press the ON/OFF button for more than 2 seconds, the temperature taken before will be displayed with an "M" in the right corner of the display.

### Replacing the battery

The battery in your thermometer needs to be replaced when "T" appears in the lower right corner of the display. Replace the battery with a 1.55V, LR41 alkaline or GP192 silver oxide type or equivalent. To replace the battery, 1. Use a cross-head screwdriver to open the battery cover. 2. Pull the battery cover away from the unit. 3. Use a non-metal pointed tool to remove the battery and insert a new battery with the positive(+) side facing down. 4. Carefully slide the battery cover back into place.

**Note:** Please keep the battery away from small children and heat.

**Warning:** The thermometer contains small parts (battery, etc.) that can be swallowed by children. Never leave the thermometer unattended. Batteries must always be kept out of reach of children and away from heating sources.

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### WARRANTY TERMS AND CONDITIONS

With respect to disposable products, ForCare Suisse warrants to the original purchaser that, at time of delivery, each standard product manufactured by ForCare Suisse shall be free from defects in material and workmanship and, when used for the purposes and indications described on the labeling, is fit for the purposes and indications described on the labeling. All warranties for a product shall expire as of the product expiration date, or if none, after two (2) years from the original date of purchase, as long as it has not been modified, altered, or misused. ForCare Suisse warranty hereunder shall not apply if: (i) a product is not used in accordance with its instructions or if it is used for a purpose not indicated on the labeling; (ii) any repairs, alterations or other work has been performed by the buyer or others on such item, other than work performed with ForCare Suisse's authorization and according to its approved procedures; or (iii) the alleged defect is a result of abuse, misuse, improper maintenance, accident or the negligence of any party other than ForCare Suisse. The warranty set forth herein is conditioned upon proper storage, installation, use and maintenance in accordance with applicable written recommendations from ForCare Suisse.

The warranty furnished hereunder does not extend to damaged items purchased hereunder resulting in whole or in part from the use of components, accessories, parts or supplies not furnished by ForCare Suisse.

The warranty does not cover battery or packaging.

### Specifications

Model No.	MT86
Measurement range	32.0°C - 42.9°C
Low temperature display	Temperature <32.0°C; display: Lo °C
High temperature display	Temperature ≥43.0°C; display: Hi °C
Display resolution	0.1°C
Full display	2 (SEC) <b>188.8°C</b>
Accuracy	35.5°C - 39.0°C: ±0.1°C; other range: ±0.2°C
Normal beep sound [ ≥37.8°C ]	30 short beep sounds in 10 seconds
Normal beep sound [ <37.8°C ]	10 long beep sounds in 10 seconds
Auto power off	8 minutes 40 seconds ± 2 minutes
Battery	LR41 or GP192
Low battery indication	If below 1.25 ± 0.05 V
Life of battery	2 years
Operating condition	Temperature: 10°C - 40°C Humidity: 30%-85% RH
Storage condition	Temperature: -10°C - 60°C Humidity: 25%-90% RH
Reference to standards	EN 12470-3, clinical thermometers; ASTM E1112; IEC 60601-1; IEC 60601-1-2 (EMC)

### Symbol Information

Symbol	Referent	Symbol	Referent
	Manufacturer		Type BF Equipment
	Consult instructions for use		Ingress protection rating
	Disposal of waste equipment		Model No.
	CE Mark		Temperature limitation
	Humidity limitation		

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