
Contents

Page

EN Instructions for Use _____	01
DE Bedienungsanleitung _____	02
FR Instructions d'utilisation _____	03
IT Istruzioni per l'uso _____	04
NL Instructies voor gebruik _____	05
ES Instrucciones de uso _____	06
PT Instruções de utilização _____	07
EL Οδηγίες χρήσης _____	08
CZ Návod k použití _____	09
RO Instrucțiuni de utilizare _____	10
AR تعليمات الاستخدام _____	11
FA دستورالعمل استفاده _____	12

ACS023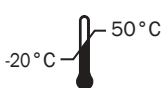

ForaCare Suisse AG

Neugasse 55, 9000 St. Gallen, SWITZERLAND
www.foracare.ch

FORA® Safety Lancets

Instructions for Use

Product Description

FORA® Safety Lancets are sterile, single-use automated devices for skin puncture. The device consists of a stainless steel lancet in a plastic housing. The lancet is manufactured from stainless steel and retracts automatically and permanently after puncture, allowing controlled length and depth of incision.

Intended Use

FORA® Safety Lancets are designed for use in the daily capillary blood collection performed by qualified personnel. They are for single-use and should only be used by adequately trained healthcare professionals in clinical settings.

Safety Information

Read the following *Safety Information* thoroughly before using the device.

- Use this device **ONLY** for the intended use described in this manual.
- Do **NOT** use accessories which are not specified by the manufacturer.
- Do **NOT** use the device if the protective tab has been removed.
- Before using the safety lancet, read all the instructions thoroughly.
- Keep the safety lancets away from young children.
- HBV, HCV, HIV and other infectious diseases can be transmitted through contact with biological samples.
- Discard all FORA® Safety Lancets in biohazard containers approved for their disposal.
- Please wear disposable gloves during venipuncture and when handling blood collection tubes.
- Do **NOT** use FORA® Safety Lancets after the expiry date.

Product Overview


Obtaining the Blood Sample


Please follow the instructions below before obtaining a drop of blood:

- Wash and dry your hands before starting.
- Select the puncture site on your fingertips.
- Rub the puncture site for about 20 seconds before penetration.

Preparing the Safety Lancet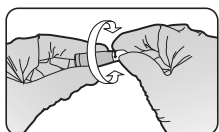

1. Twist and remove the cap.

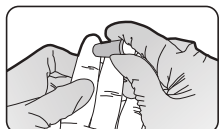2. Press the lancet gently against the test site.

3. Gently squeeze the finger to obtain a drop of blood.

4. Discard used lancet in a puncture-resistant container.

* Do not use if the protective cap has been removed.

Important Information

- Storage condition: -20°C~50°C (-4°F~122°F), below ≤80% relative humidity.
- Store the safety lancets in a cool and dry place. Keep away from direct sunlight and heat.
- Use each safety lancet immediately after removing the protective cap.
- Do not use the safety lancets beyond the expiry date.
- Keep the safety lancets away from children since they may be choking hazards.

Symbol Information

Symbol	Referent	Symbol	Referent
	Consult instructions for use		Storage/ Transportation temperature limitation
	Use by		CE Mark
	Batch code		Manufacturer
	Keep away from sunlight		Dispose of the packaging properly after use
	Keep dry		Caution, consult accompanying documents
	Sterilized using irradiation		Do not use if package is damaged
	Do not reuse		Storage/ Transportation humidity limitation

Specification

Dimensions : 42.5 (L) x 15.6 (W) x 11.2 (H) mm
Weight : 2.5g

Features

- Tri bevel sharpened point
- Automated lancet
- Patented internal dual spring mechanism
- Unique ergonomic design
- Gamma sterilized, 5 years shelf life

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